

Snappy Abutment™ 4.0 and 5.5

QUICK GUIDE* FOR IMPRESSION TECHNIQUE

This Quick Guide does not replace the Instructions for Use. Please review the instructions for use included with the products.

Step 1

- Remove the abutment and abutment screw from the packaging. Place the Screwdriver Unigrip into the abutment screw access hole. Holding the abutment screw, abutment and Screwdriver Unigrip as illustrated to the right, place the Snappy Abutment. Engage the implant connection and hand-tighten the abutment screw using the Screwdriver Unigrip.
- Take a radiograph to verify proper seating.
- Tighten the abutment screw to 35 Ncm using the Manual Torque Wrench Prosthetic (please see instructions for use for this item) and Screwdriver Machine Unigrip.

Step 2

Impression Process:

- Align the arrow on top of the impression coping with the facial groove on the abutment. Push the impression coping onto the abutment. A “snap” will indicate that the impression coping is fully engaged and well adapted to the margin of the abutment.
- Note: If the Snappy Abutment has been modified, do not use the impression cap. Take a standard C&B impression.

Step 3

- With the impression coping in place, inject a heavy body impression material (polyether material or polyvinylsiloxane) around the impression coping and into the tray.
- Record the impression. When the impression is removed, the impression coping will disengage from the Snappy Abutment and gets picked up in the impression.

Step 4

- Replace the healing cap or provisionalize the Snappy Abutment chairside using the plastic/temporary coping** and a fabricated provisional restoration.
- Send the impression and the Snappy Abutment Replica (sold separately) to the lab for crown fabrication.




* Consult the appropriate procedures manual for detailed instructions.

** Warning: Do not use Plastic/Temporary Coping Snappy Abutment with polyurethane cements. The cement will not cure!

Product selection for the Snappy Abutment™ 4.0 and 5.5

Snappy Abutment™ 4.0 package

4.0 articles included in package

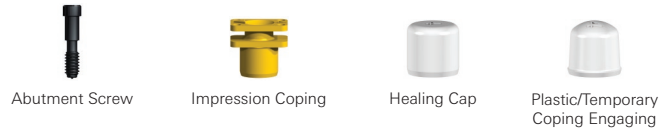

Internal tri-channel connection (NobelReplace™)		
NP 0.75 mm		35962
NP 1.5 mm		35966
RP 0.5 mm		35963
RP 1.5 mm		35967
WP 0.5 mm		35964
WP 1.5 mm		35968
6.0 0.5 mm		35965
6.0 1.5 mm		35969



Internal conical connection (NobelActive™)		
NP 1.5 mm		35980
NP 3.0 mm		35981
RP 1.5 mm		35978
RP 3.0 mm		35979
RP Wide 1.5 mm		35976
RP Wide 3.0 mm		35977



External hex connection (Brånemark System®)		
NP 1.0 mm		35970
NP 2.0 mm		35973
RP 1.0 mm		35971
RP 2.0 mm		35974
WP 1.0 mm		35972
WP 2.0 mm		35975

Additional Snappy Abutment™ 4.0 products (sold separately)




Snappy Abutment™ 4.0 Abutment Replica		
NP		35982
RP		35983
NRpl WP/NActv RP Wide		35984
NRpl 6.0/Bmk WP		35985



Snappy Abutment™ 4.0 Plastic/Temporary Coping Non-Engaging		
NP/RP		35989
NRpl WP/NActv RP Wide		35990
NRpl 6.0/Bmk WP		35991

Snappy Abutment™ 5.5 package

5.5 articles included in package

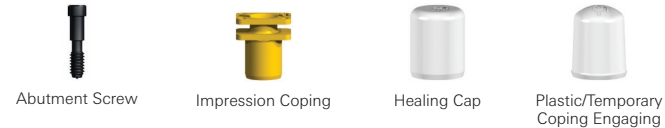

Internal tri-channel connection (NobelReplace™)		
NP 0.75 mm		35798
NP 1.5 mm		35802
RP 0.5 mm		35799
RP 1.5 mm		35803
WP 0.5 mm		35800
WP 1.5 mm		35804
6.0 0.5 mm		35801
6.0 1.5 mm		35845



Internal conical connection (NobelActive™)		
NP 1.5 mm		35946
NP 3.0 mm		35947
RP 1.5 mm		35852
RP 3.0 mm		35853
RP Wide 1.5 mm		35960
RP Wide 3.0 mm		35961



External hex connection (Brånemark System®)		
NP 1.0 mm		35846
NP 2.0 mm		35849
RP 1.0 mm		35847
RP 2.0 mm		35850
WP 1.0 mm		35848
WP 2.0 mm		35851

Additional Snappy Abutment™ 5.5 products (sold separately)



Snappy Abutment™ 5.5 Abutment Replica		
NP		35854
RP		35855
NRpl WP/NActv RP Wide		35856
NRpl 6.0/Bmk WP		35857



Snappy Abutment™ 5.5 Plastic/Temporary Coping Non-Engaging		
NP/RP		35861
NRpl WP/NActv RP Wide		35862
NRpl 6.0/Bmk WP		35863



Notes

– Use only 4.0 components for Snappy Abutment™ 4.0 and 5.5 components for Snappy Abutment™ 5.5.

*Consult the appropriate procedures manual for detailed instructions.

Product images are not necessarily to scale.

In order to improve readability, Nobel Biocare does not use TM/® in running text. In so doing, however, Nobel Biocare does not waive any right to the trademark or registered mark and nothing herein shall be construed to the contrary.